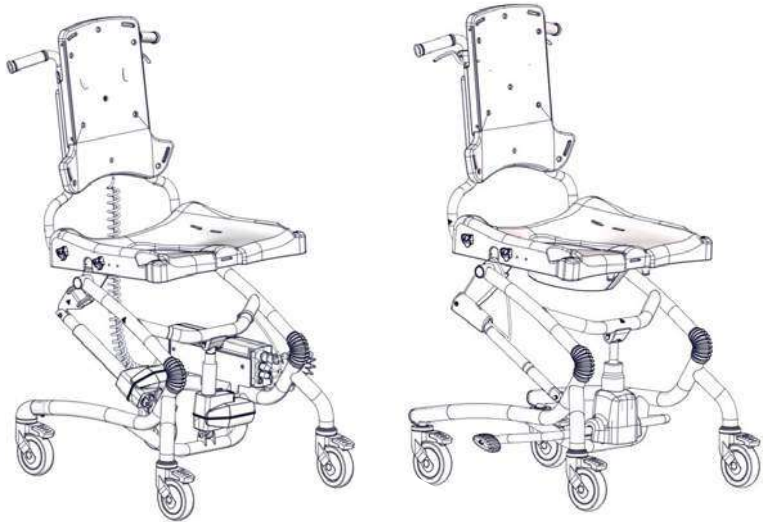
≤ 100 kg (220 lb)

≤ 130 kg (286½ lb)

Heron User Guide

EN 12182


2019-12-rev.15

	Hydraulic	Power		
	3	13	ENGLISH DEUTSCH	32 - 39 40 - 49
		14	NEDERLANDS DANSK	50 - 57 58 - 65
	4 - 5	15	NORSK SVENSKA	66 - 73 74 - 81
	6	16	SUOMI ESPAÑOL	82 - 89 90 - 97
	7	17	FRANÇAIS ITALIANO	98 - 105 106 - 113
	8	18	PORTUGUÊS PORTUGUÊS DO BRASIL	114 - 121 122 - 129
	9	19	РОССИЯ POLSKI	130 - 137 138 - 145
	10	20	ČESKY 中國	146 - 153 154 - 160
		21	日本 ΕΛΛΗΝΙΚΗ	161 - 168 169 - 176
		22 - 23		
	11	24		
	12	25		
	26 - 29			
mm (inch)	30			
	31			


A.

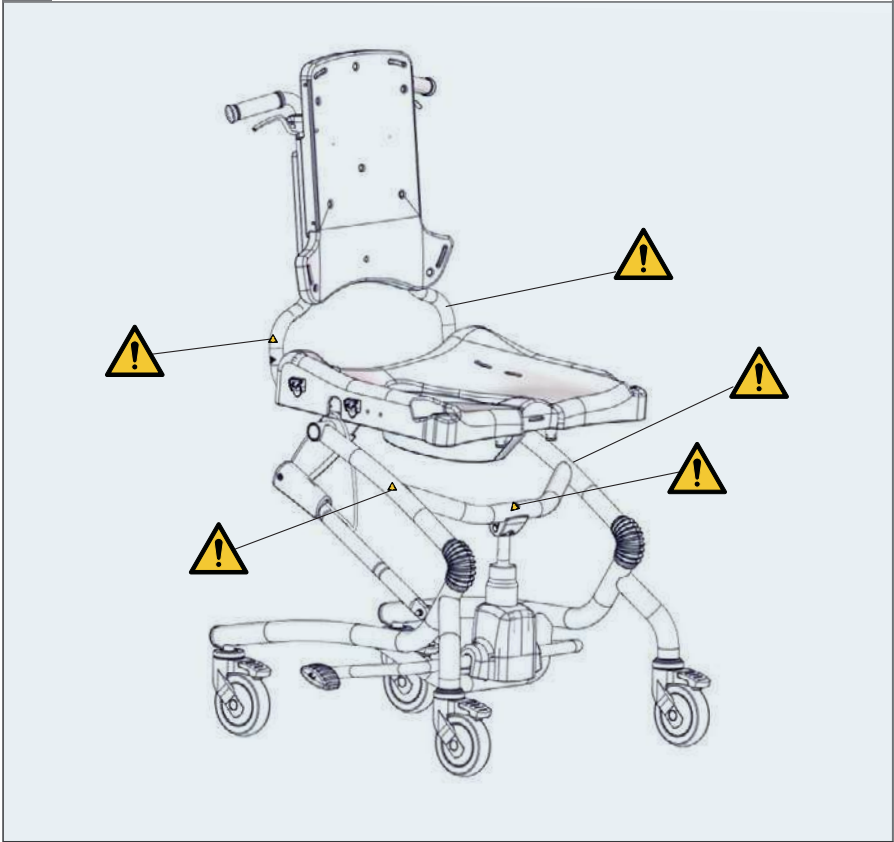 05

B.


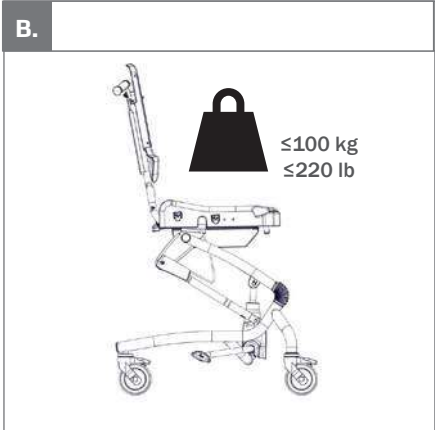
1.

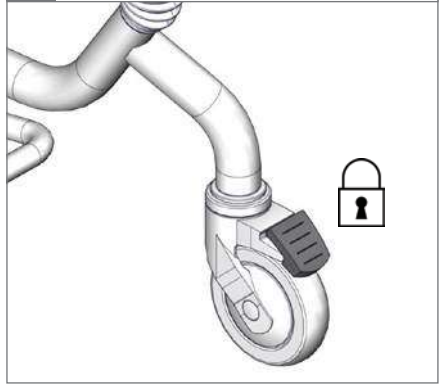2.




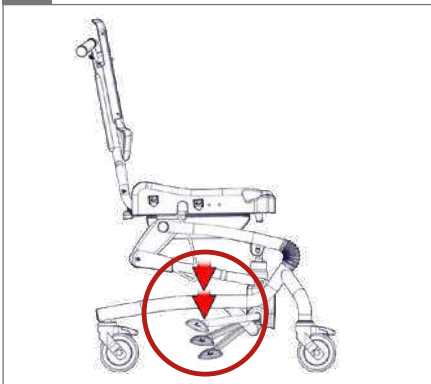
3.

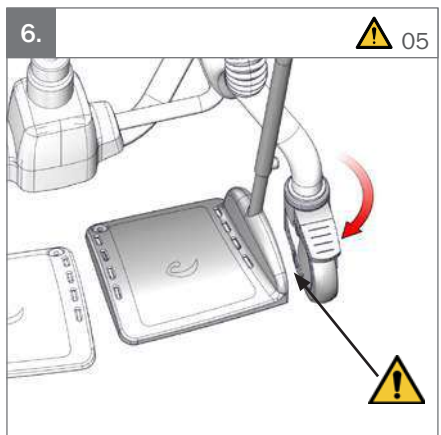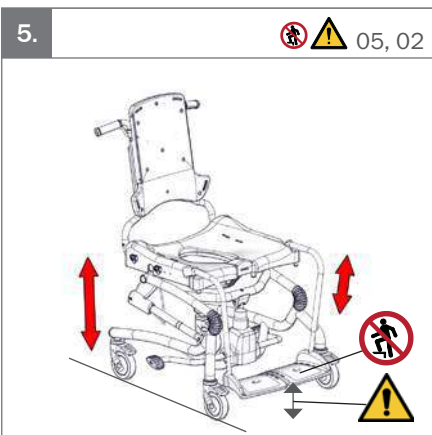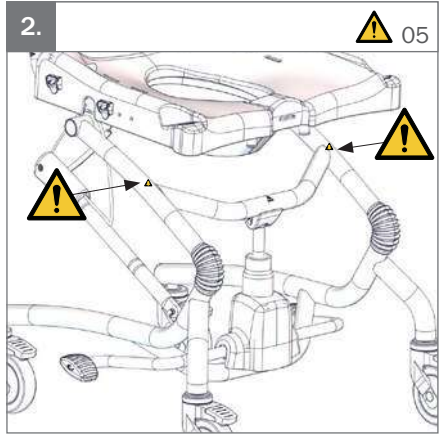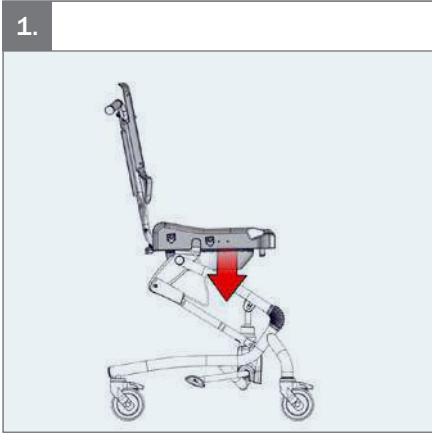4.


1.


-5° -> 25°

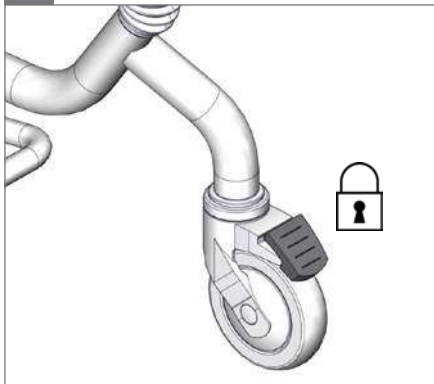
2.

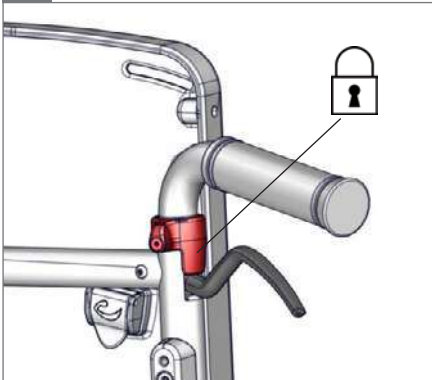
3.

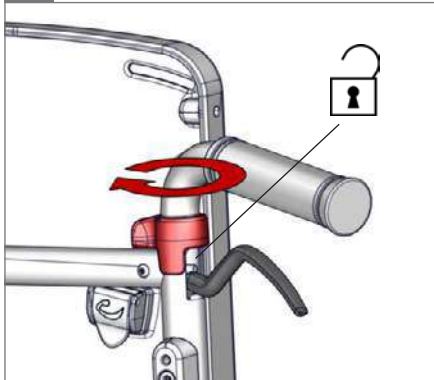
4.





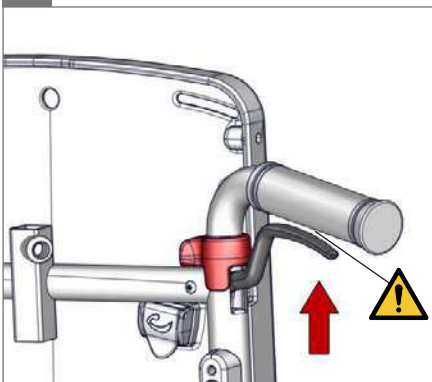
5.


 05

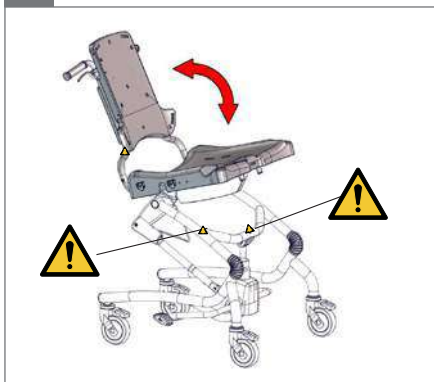6.


 05

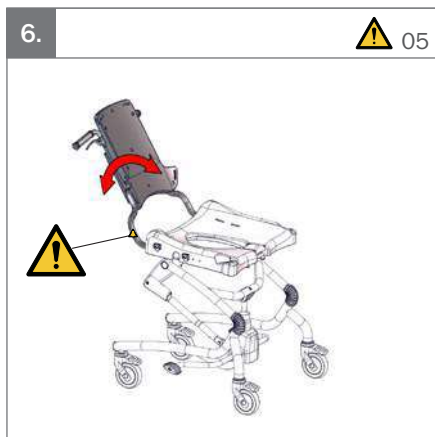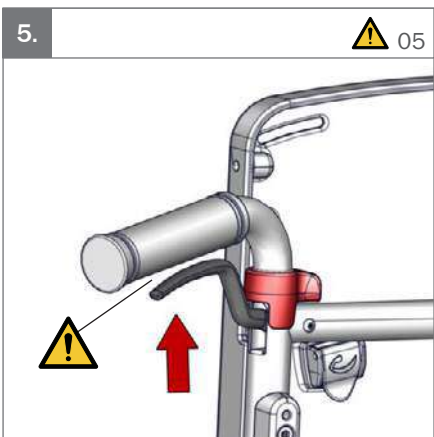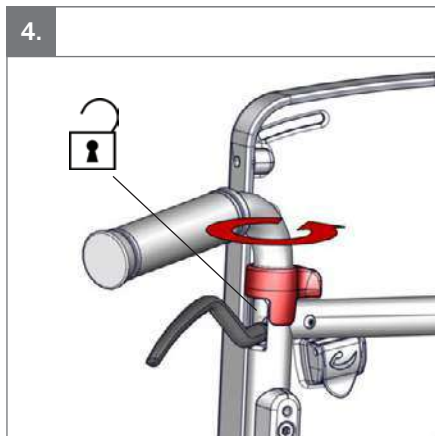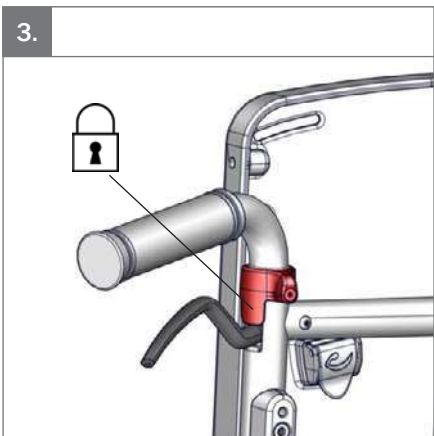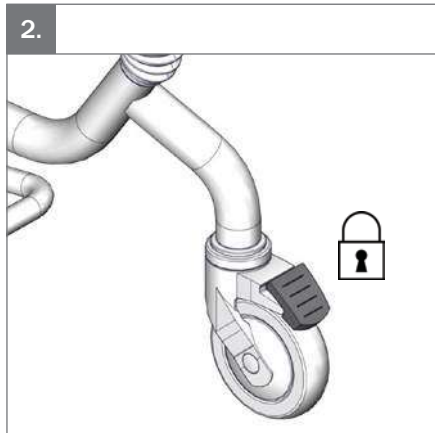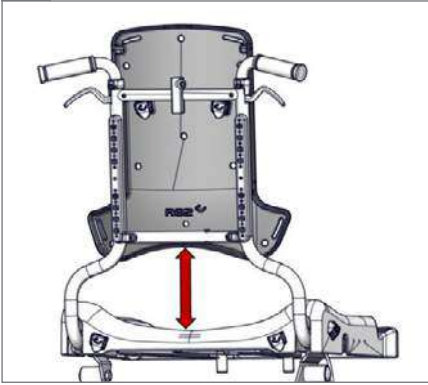1.

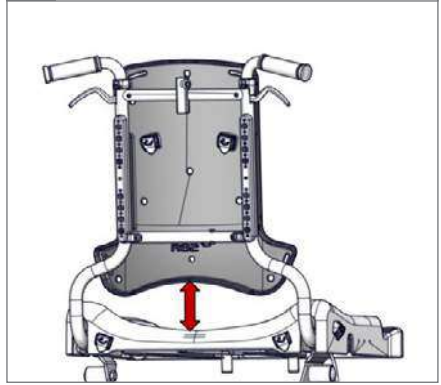2.


3.


○ 5 mm

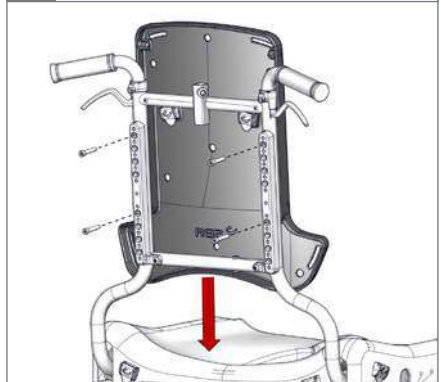4.





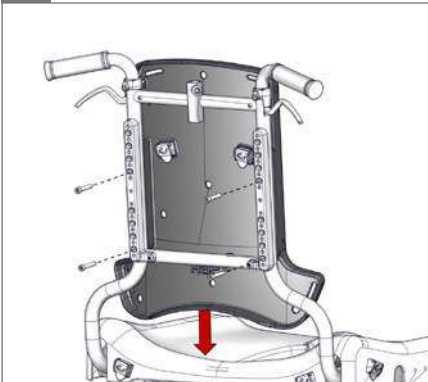
5.

6.

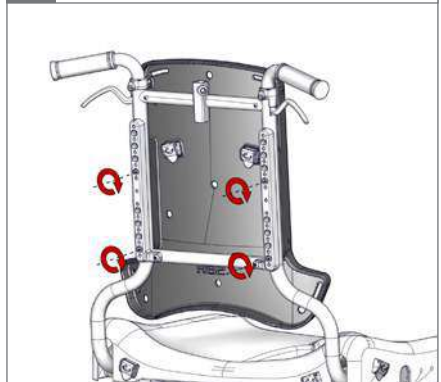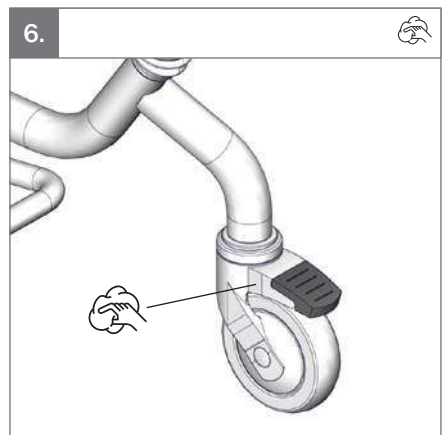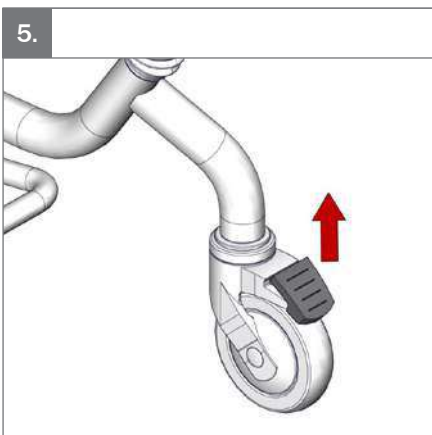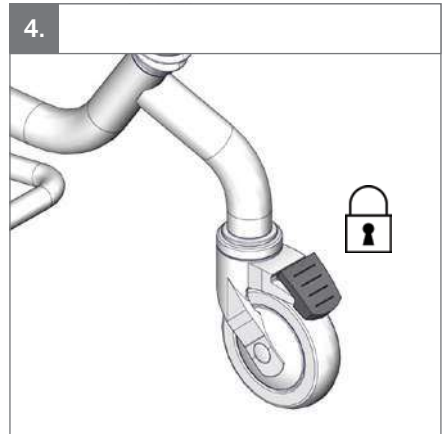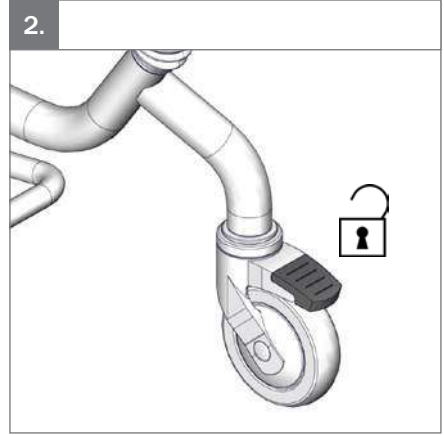○ 5 mm - 2,5 Nm



1.

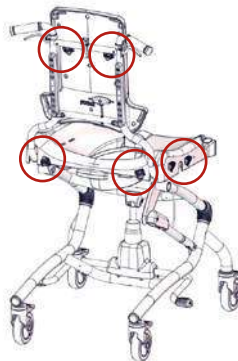
2.

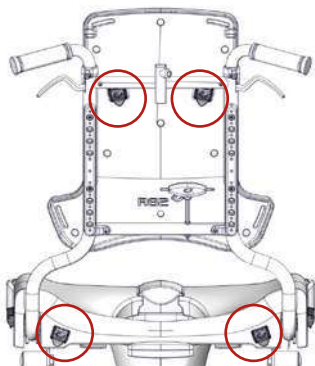
3.

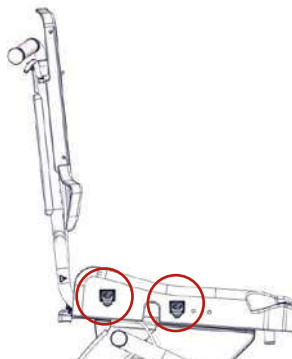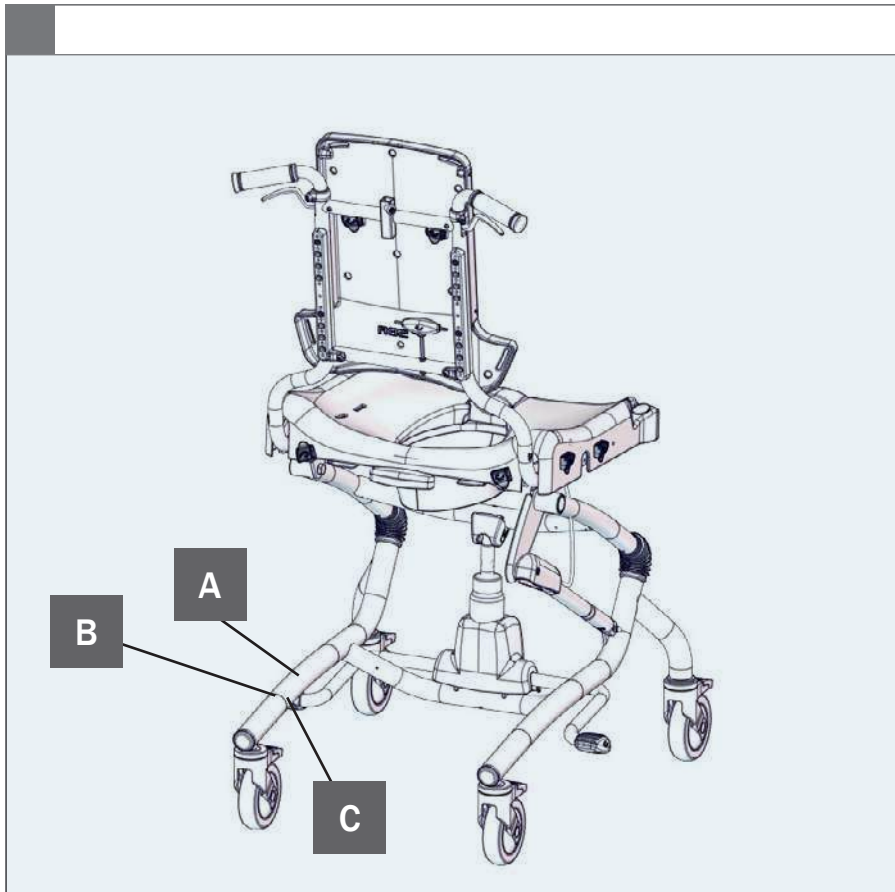4.

A

XXXXXXXXX Max load: xxx kg
 DMR ver.:XXXX
 Art. no.:XXXXXXXX
 Product:XXXX - Size X

B




R82[®]

Parallevej 3
 DK-8751 Gedved
 R82.org
 Made in XXXXXXXX YYY-MM-DD

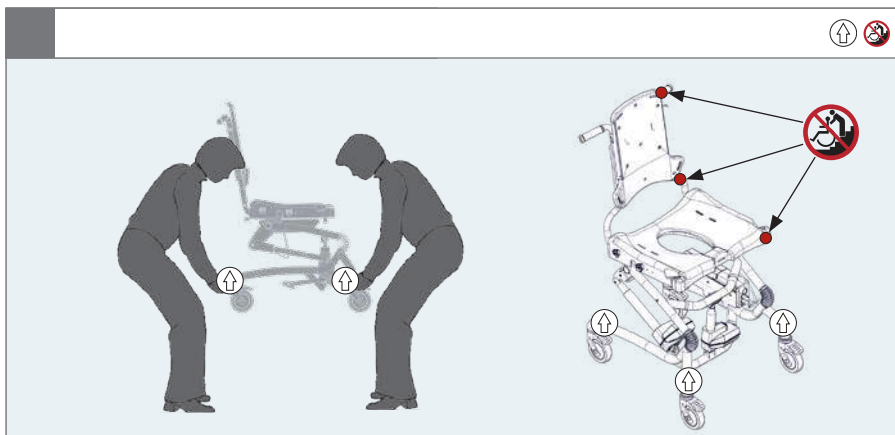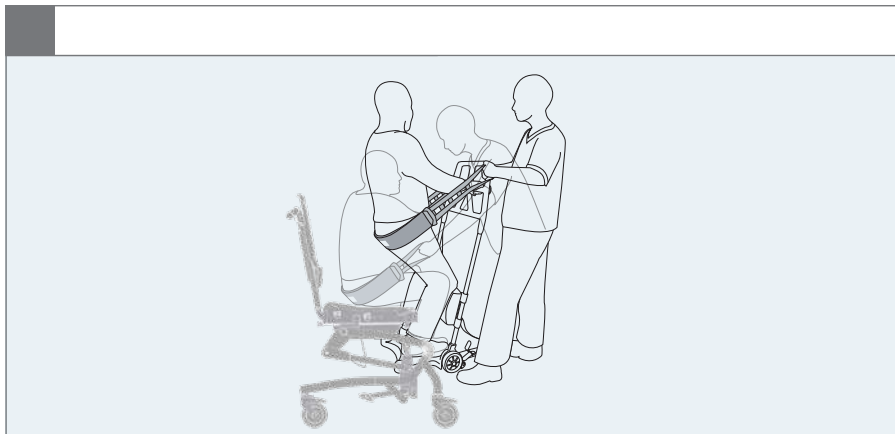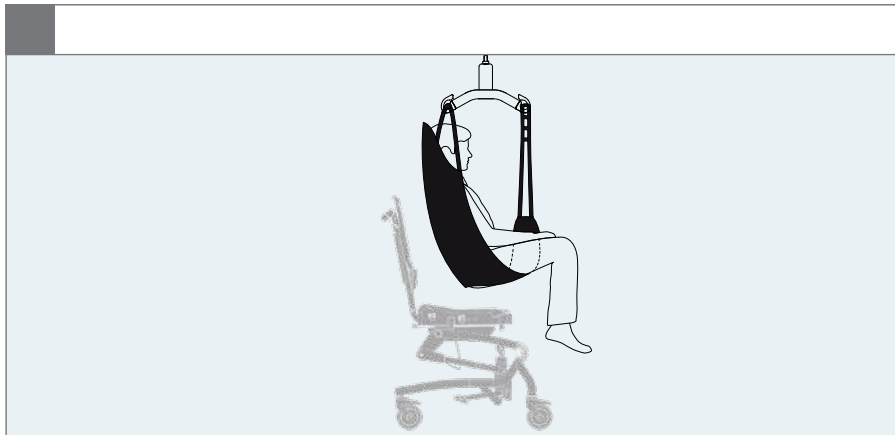C



A.


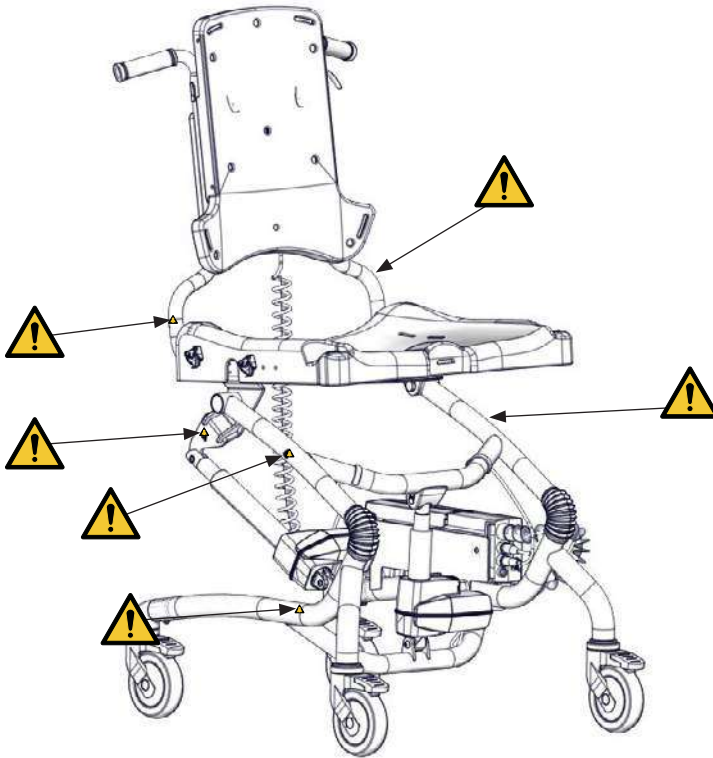
! 05




B.

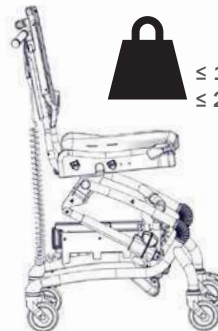≤ 130 kg
≤ 286½ lb

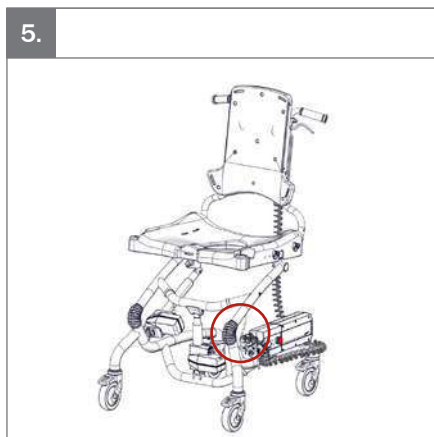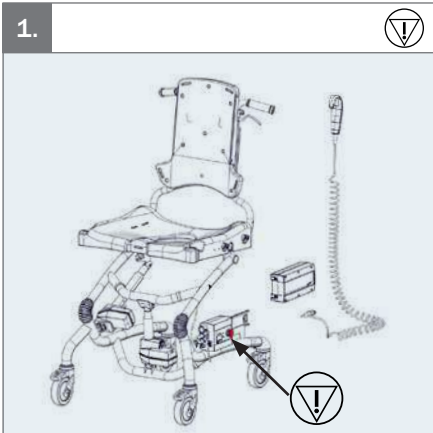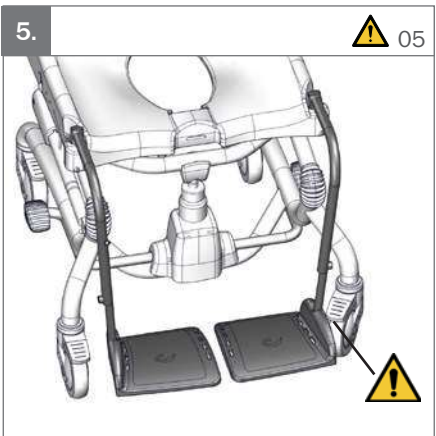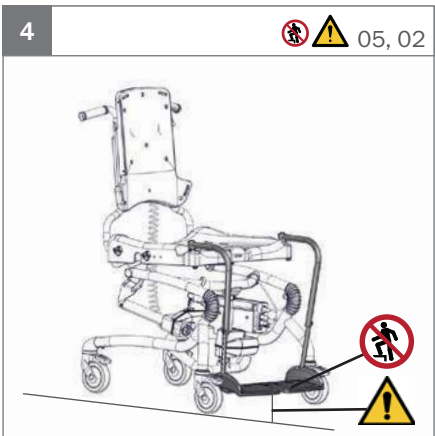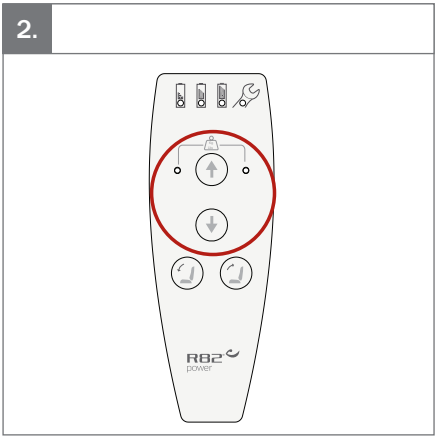
1.


-5° -> 25°

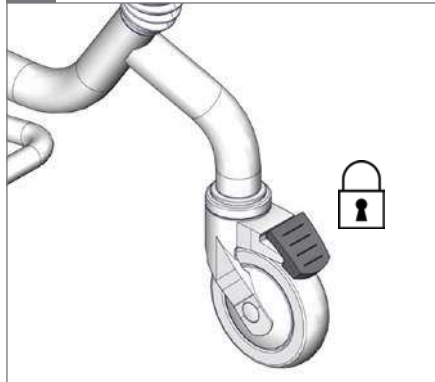2.

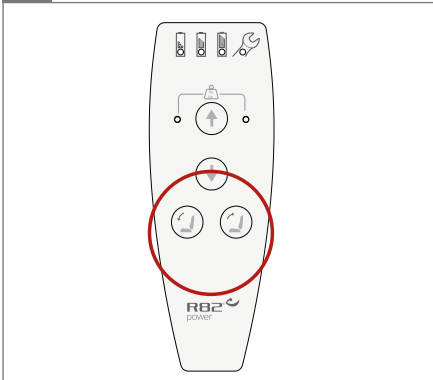3.

4.

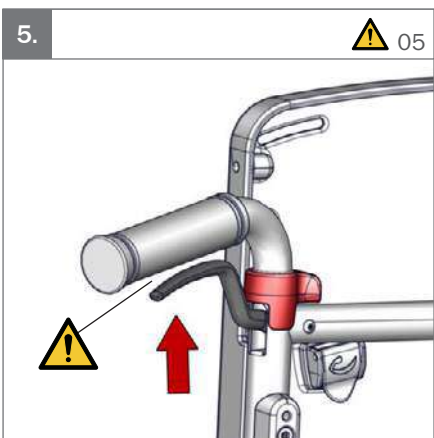
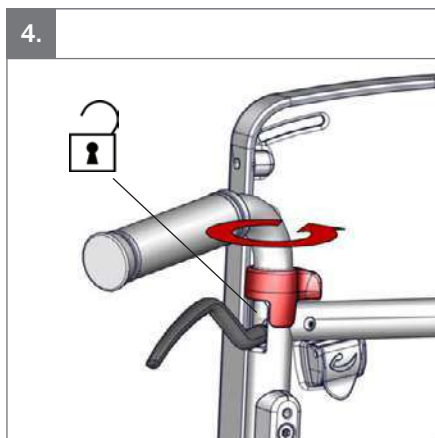
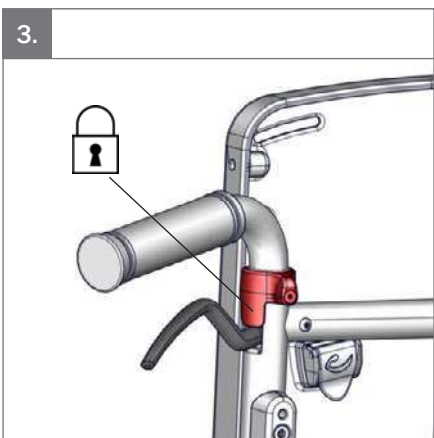
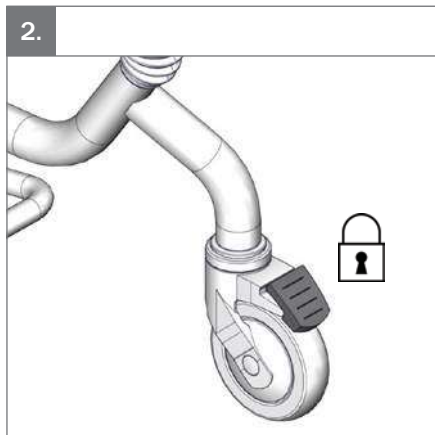
5.


6.

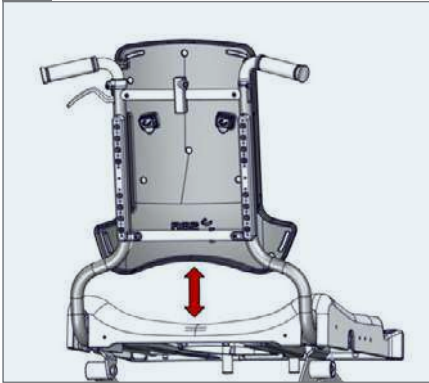
1.

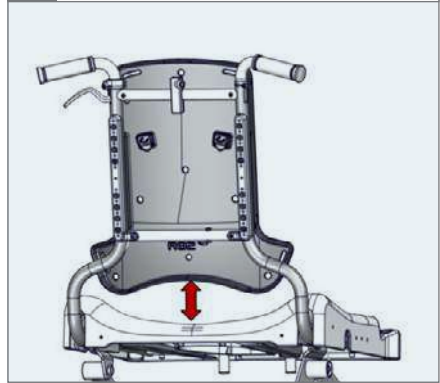
2.


3.

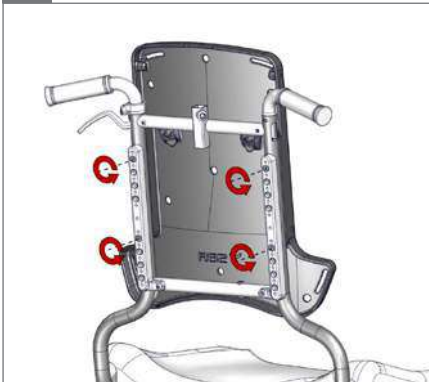○ 5 mm

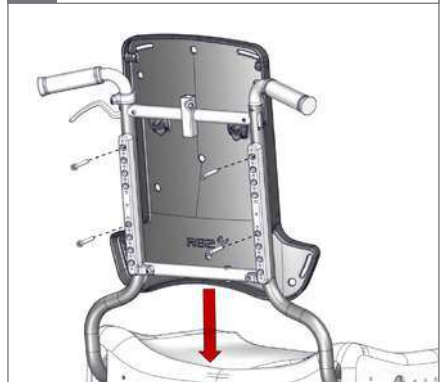4.





5.




6.

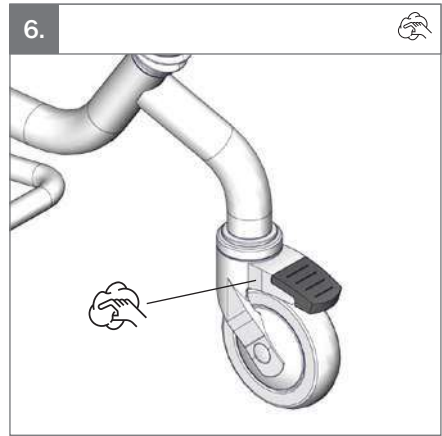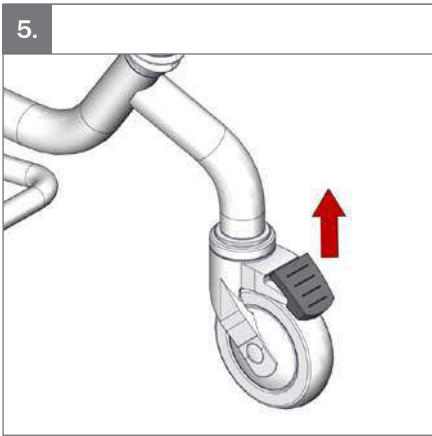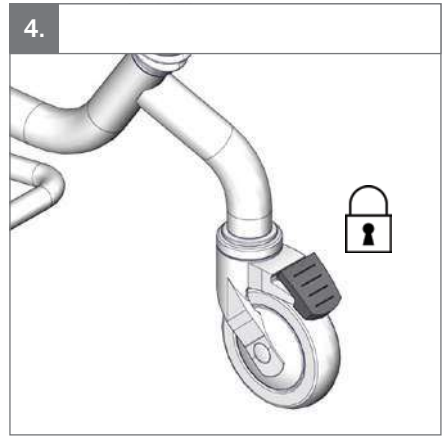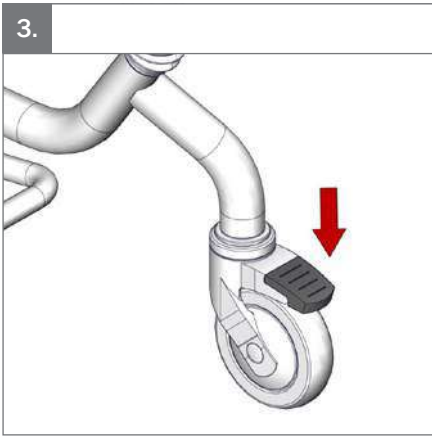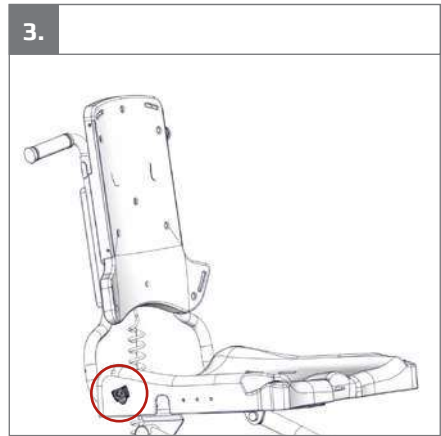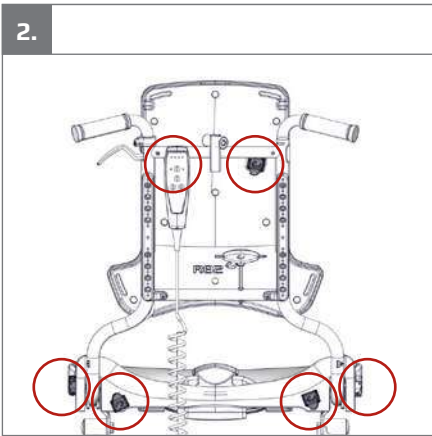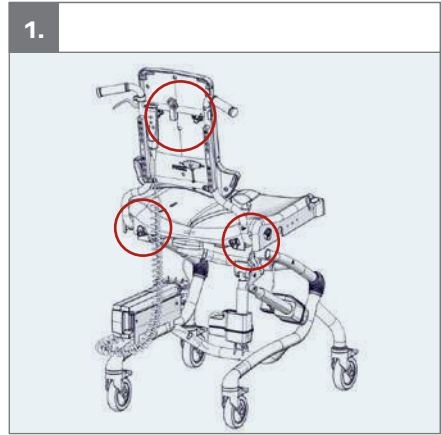○ 5 mm - 2,5 Nm


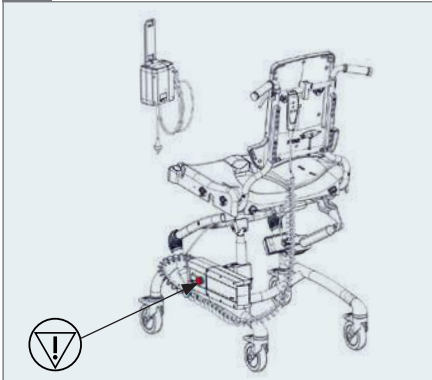
1.


2.

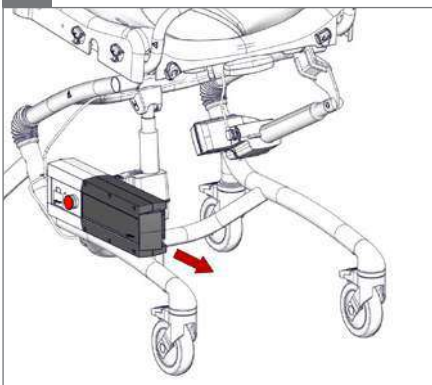
3.

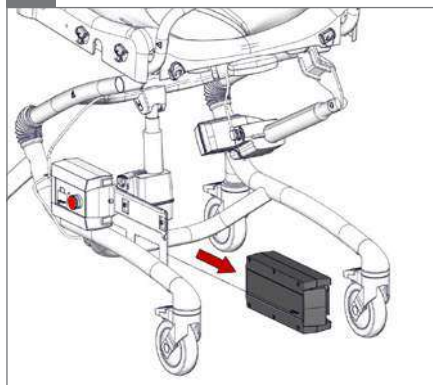4.

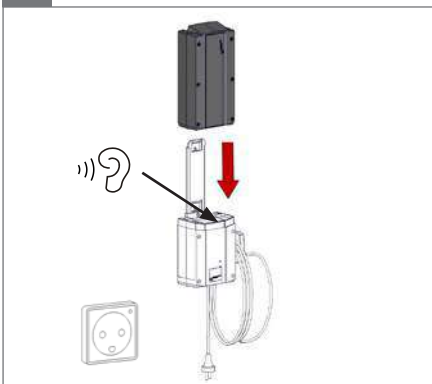5.

6.

100 - 240 V / 50-60 Hz


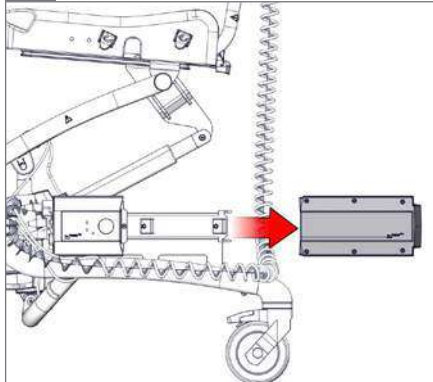
1.

2.

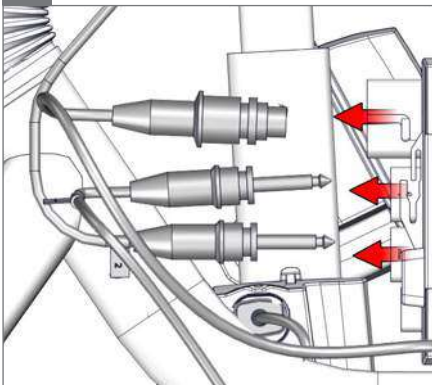
3.


4.

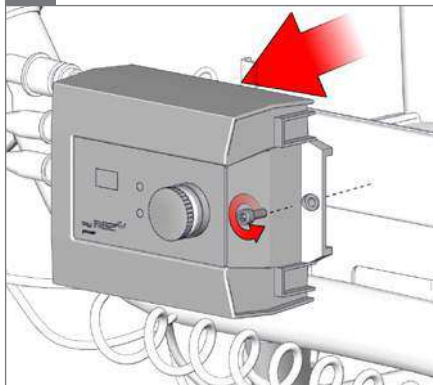Ø 3 mm

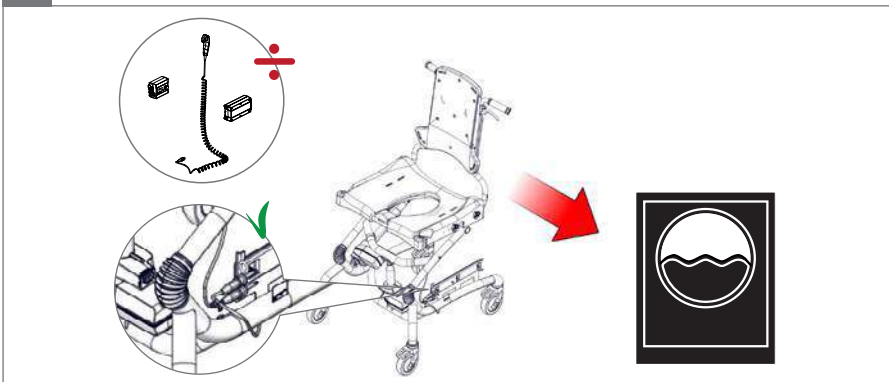
5.


7.

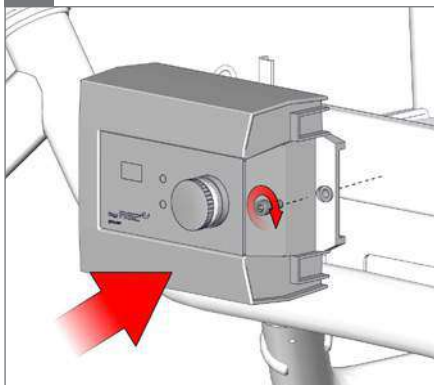
⌀ 3 mm

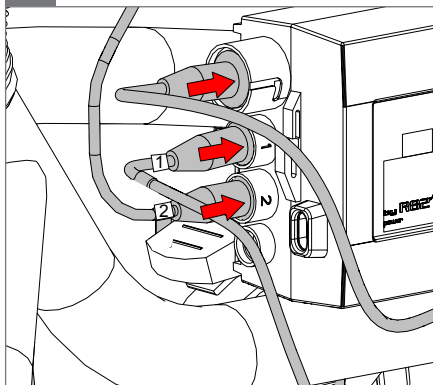8.

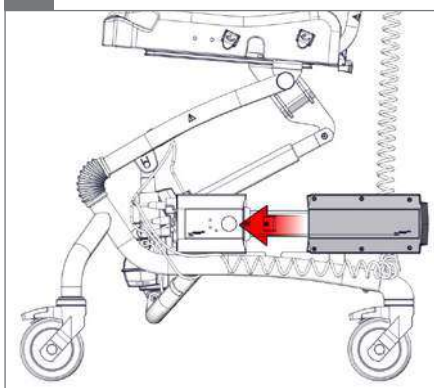
9.

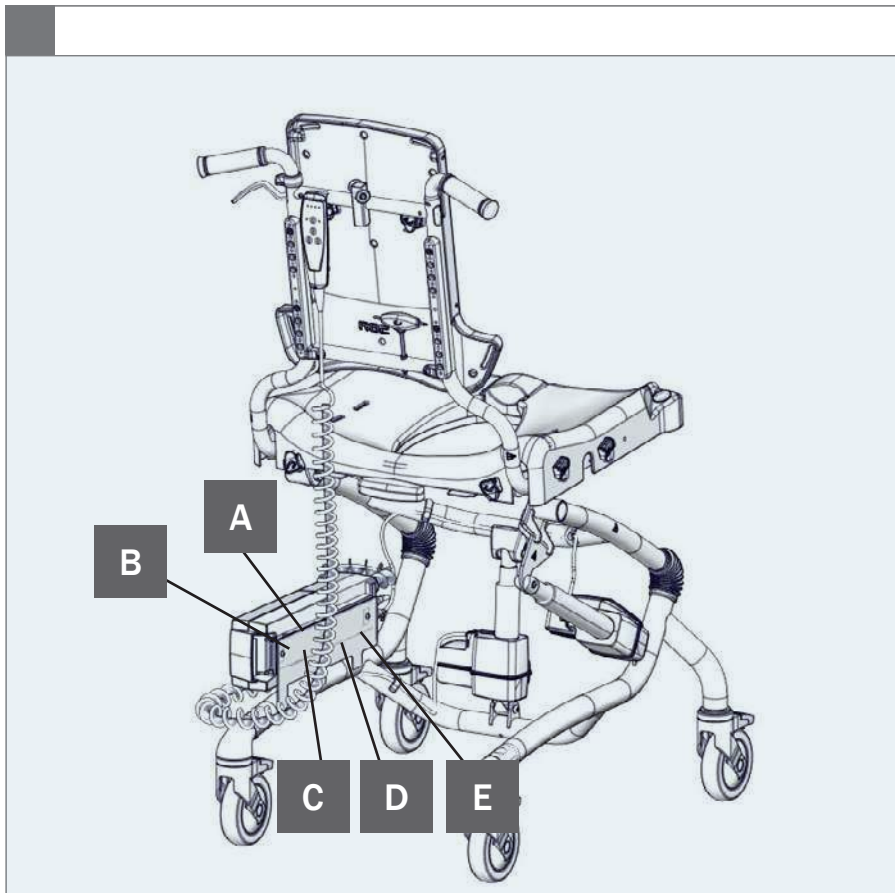10.

A

Max load: xxx kg
 DMR ver.:XXXX
 Art. no.:XXXXXXX
 Product:XXXX - Size X

B

Parallevej 3
 DK-8751 Gedved
 R82.org
 Made in XXXXXXXX YYY-HM-DD

C




D

Storage & Transport

Operating

E

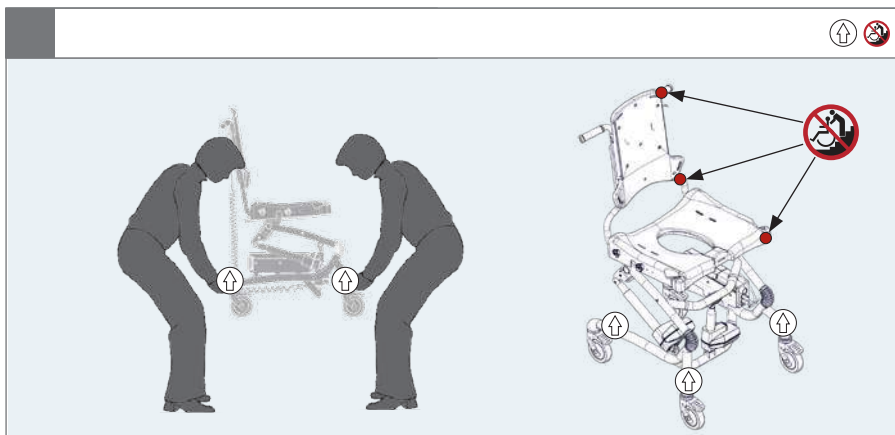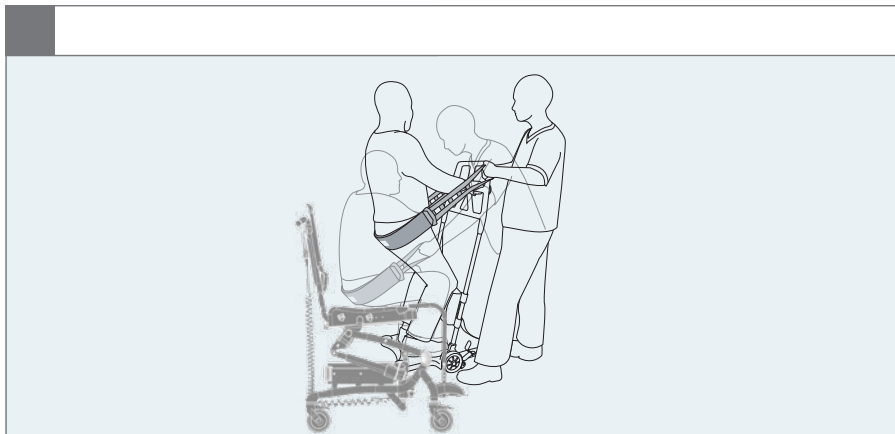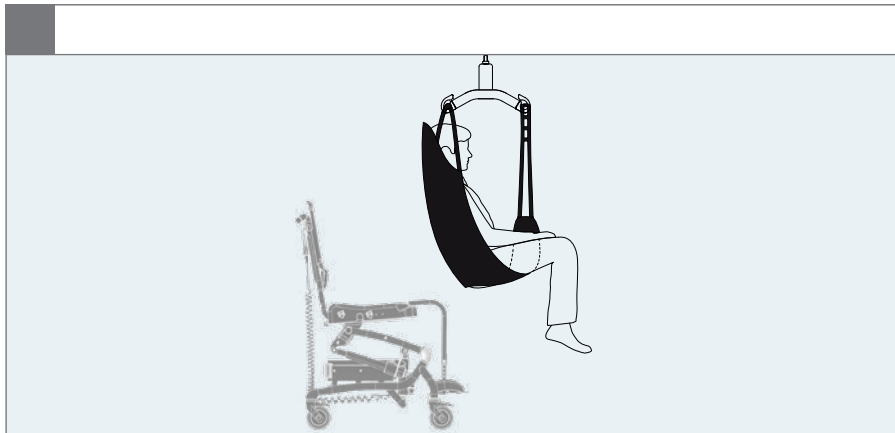IP 24

**EN:** For indoor use**NO:** For innendørs bruk**FR:** Utilisation intérieure**RU:** Для использования в помещениях**JP:** 屋内使用**DE:** Für den Innengebrauch**SV:** För inomhusbruk**IT:** Per uso interno**PL:** Do użytku wewnątrz**GR:** Κατάλληλο για εσωτερική χρήση**NL:** Voor binnenhuis gebruik**FIN:** Sisäkäyttöön**PT:** Para utilização em interiores**CZ:** Pro vnitřní používání**DK:** Til indendørs brug**ES:** Para uso en interiores**BR:** Para uso interno**SN:** 适用于室内使用**EN:** Consult instructions. The latest version of the instructions is always available on the R82 website and can be printed in larger sizes**NO:** Sjekk bruksanvisning på nett. Nyeste versjon av bruksanvisningen er alltid tilgjengelig, og kan skrives ut fra R82 nettsted**FR:** Se reporter aux instructions / consulter les instructions. La version la plus récente est toujours disponible et peut être imprimée dans un grand format à partir du site internet de R82**RU:** Прочитайте инструкции. Новейшая версия инструкций всегда доступна на веб-сайте компании R82 и может быть распечатана в увеличенном размере**JP:** 説明をお読みください。この説明の最新バージョンは、R82のWebサイトからいつでもご利用いただけます。大きなサイズで印刷することができます**DE:** Bedienungsanleitung. Die neueste Version der User Guide ist auf der R82 Internetseite verfügbar und steht zum Download bzw. Ausdruck zur Verfügung**SV:** Se anvisningarna. Den senaste versionen av anvisningarna finns alltid tillgänglig och kan skrivas ut i större storlek från R82:s webbplats**IT:** Leggere le istruzioni. Le ultime versioni dei manuali di istruzione sono sempre disponibili sul sito web R82 e possono essere stampate in dimensioni maggiori**PL:** Zapoznanie się z instrukcją. Najnowsze wersje instrukcji obsługi są dostępne na stronie internetowej R82 i można je wydrukować w większych formatach**GR:** Διαβάστε τις οδηγίες. Οι τελευταίες εκδόσεις των οδηγιών είναι πάντοτε διαθέσιμες στην ιστοσελίδα της R82 και μπορούν να εκτυπωθούν σε μεγαλύτερα μεγέθη**NL:** Raadpleeg de instructies. De meest recente versie van alle instructies zijn altijd beschikbaar en kunnen worden afgedrukt in de groter formaat vanaf de R82 website**FIN:** Lisäohjeet. Viimeisin versio kaikista ohjeista on aina saatavilla ja tulostettavissa R82 nettisivustolta**PT:** Leia as instruções. As últimas versões das instruções estão sempre disponíveis na página da R82 na internet e podem ser impressas em tamanhos maiores**CZ:** Poradenství a instrukce. Nejaktuálnější verze všech instrukcí je vždy dostupná na webu R82, odkud se dá stáhnout a i vytisknout**DK:** Se vejledning. Den seneste version af denne vejledning er altid tilgængelig og kan udskrives i større størrelser fra R82 hjemmeside**ES:** Consulte las instrucciones. La última versión de todas las instrucciones está siempre disponible y se puede imprimir en tamaños superiores desde la página web de R82**BR:** Leia as instruções. As versões mais recentes das instruções sempre encontram-se disponíveis no site da R82 e podem ser impressas em tamanhos maiores**SN:** 参考说明。所有说明的最新版都能够R82网络上下载或打印**EN:** Emergency stop**NO:** Nødstop**FR:** Arrêt d'urgence**RU:** Экстренная остановка**JP:** 非常停止**DE:** Not-Aus**SV:** Nödstop**IT:** Stop di emergenza**PL:** Awaryjne zatrzymanie**GR:** Διακοπή έκτακτης ανάγκης**NL:** Noodstop**FIN:** Häätäpysäytys**PT:** Paragem de emergência**CZ:** Nouzový stop**DK:** Nødstop**ES:** Parada de emergencia**BR:** Parada de emergência**SN:** 紧急停止

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
mm (inch)


 Overall width	670 mm (26½")	670 mm (26½")
 Length	710 mm (28")	710 mm (28")
 Overall height	1080–1350 mm (44½–53¼")	1130–1350 mm (44½–53¼")
 Effective seat width	550 mm (21¾")	550 mm (21¾")
Seat width between hip supports (accessories)	350–490 mm (13¾ - 19¼")	350–490 mm (13¾–19¼")
 Effective seat depth	450 mm (17¾")	450 mm (17¾")
 Back width, top of backrest	290 mm (11½")	290 mm (11½")
 Backrest height	555–625 mm (21¾–24½")	555–625 mm (21¾–24½")
 Seat surface height at front edge	510–780 mm (20–30¾")	560–780 mm (22–30¾")
 Seat plane angle	-5–25°	-5–25°
 Backrest angle	0–30°	0–30°
 Lower leg length	400–560 mm (15¾–22")	400–560 mm (15¾–22")
 Minimum turning radius	570 mm (22½")	570 mm (22½")
 Total mass	28 kg (61.75 lb)	31 kg (68.25 lb)
 Max load	100 kg (220 lb)	130 kg (286.6 lb)
 Static stability sideways	5°	5°
 Static stability downhill	10°	10°
 Static stability uphill	10°	10°

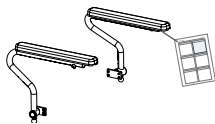
9996097103


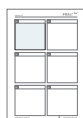
9996097104


9996097105

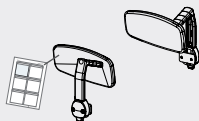
9996097106

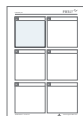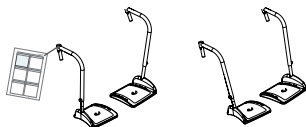9996097107

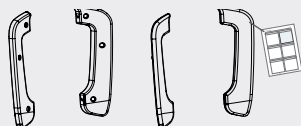
9996097108

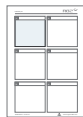9996097109

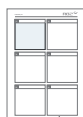9996097110

9996097111

ENGLISH

We appreciate your choice of a new product from R82 - a worldwide supplier of technical aids and appliances for children and teenagers with special needs. To fully benefit from the options offered by this product please read this User Guide before use and save it for future reference.

Intended purpose

The Heron is a toilet/bathing chair suitable for larger children and youngsters. It is adjustable in angle and height and assist special need users with their seating posture. The Heron can be placed over the toilet or can be used with a pan. The Heron comes in one size and is only for indoor use but can in an emergency be taken outside.

Enclosed tools:

- 3+4+5 mm Allen key

Declaration of Conformity

This product conforms to the requirements of the Medical Devices Directive (93/42/EEC). The CE mark must be removed when rebuilding the product, when used in combination with a product from another manufacturer or when using other than original R82 spare parts and fittings. Further, this product conforms to the requirements according to:

- EN 12182

Disposal

When a product reaches the end of its lifetime, it should be separated by material type so that the parts can be recycled or disposed of properly. If required, please contact your local dealer for a precise description of each material. Please contact your local authorities to be informed of the options available for recycling in an environmentally-friendly way.

Accessories and spare parts

The products from R82 can be supplied with a variety of accessories which comply with the needs of the individual user. Spare parts can be ordered on request. Find the specific accessories on our website or contact your local dealer for further information.

R82 Warranty

R82 offers a 2-year warranty against defects in workmanship and materials and a 5-year warranty on breakage of the metal frame caused by defects in welds. The warranty would be adversely affected if the customer's responsibility of servicing and/or maintenance is not carried out according to the guidelines and intervals prescribed by the supplier and/or stated in the User Guide. For further information, we refer to the R82 homepage/download.

The warranty is only granted in the country where the product was purchased and if the product can be identified by the serial number. The warranty does not cover accidental damage, including damage caused by misuse or neglect. The warranty does not extend to consumable items e.g. tires or upholstery, which are subject to normal wear and tear and need periodic replacement.

The warranty is null and void if other than original parts/accessories are used, or if the product has not been maintained, repaired or altered according to the guidelines and intervals prescribed by the supplier and/or stated in the User Guide. R82 reserves the right to inspect the product being claimed for and the relevant documentation before agreeing to the warranty claim, and to decide upon whether to replace or repair the defective product. It is the customer's responsibility to return the item being claimed for under warranty to the address of purchase.

Safety

Signs, symbols and instructions placed on this product must never be covered or removed and must remain present and clearly legible throughout the entire lifetime of the product. Immediately replace or repair illegible or damaged signs, symbols and instructions. Contact your local dealer for instructions.

In case of an adverse event occurred in relation to the device, incidents should be reported to your local dealer and the national competent authority in a timely manner. The local dealer will forward information to manufacturer.

Caregiver

- Read the "User Guide" carefully before use and keep it for future reference. Incorrect use of this product may cause serious injury to the user
- Always use the correct lifting techniques and aids for this purpose
- Never leave the user unattended in this product. Ensure permanent supervision by an adult
- Repair/replacements must only be made using new original R82 spare parts and fittings and carried out in accordance with the guidelines and service intervals prescribed by the supplier
- If there is any doubt as to the continued safe use of your R82 product or if any parts should fail, stop using the product immediately and contact your local dealer as soon as possible

Environment

- Observe the surface temperature on the product, before placing an user in it. This goes especially for users with insensitive skin as they cannot feel the heat. If the surface temperature is above 41 °C then allow the product to cool before use
- Use this product on a level surface without any obstacles. Be aware of reduced maneuverability on slippery surfaces
- When using on inclines, please refer to measurements

User

- If the user is close to the max. load limit and/or has a lot of involuntary movements, e.g. rocking, a larger size with a higher max. load or a different R82 product should be considered

Product

- Carry out all positional adjustments on the product and accessories and ensure that all knobs, screws and buckles are securely fastened before use. Keep all tools out of the reach of children
- Activate the brakes before placing the user into the product
- Ensure the stability of the product is maintained before placing the user in it
- We recommend you notify the user before angling the seat and back
- Before moving the chair, with or without a user seated, please ensure the seat is in horizontal position, the back in vertical position. Do not move an angled seat
- Ensure the castors are fully-functional and secure before use
- Inspect the product and all of its accessories and replace any worn out parts before use
- Do not expose the gas spring to pressure or high temperatures. DO NOT puncture
- Do not expose the piston rod to scratches etc. It is important to use a cloth or the like, if tools are used to adjust the gas spring
- The emergency stop is activated when using the red button on the control box

Service information

Maintenance of a medical device is wholly the responsibility of the owner of that device. Failure to maintain a device in accordance with the instructions may invalidate the device's warranty. Furthermore, failure to maintain a device may compromise the clinical condition or safety of users and/or their caregivers. Do not perform service and maintenance while the user is in the product.

1st Day

- Read the User Guide carefully before use and store it in a safe place for future reference

Before every use

- Check that all castors are moving freely and that all wheel locks/brakes are easily activated
- Make sure all cables are securely plugged in before use in the shower
- Inspect all cables for cracks or signs for worn-out parts. Never use a product which appears to be substandard or faulty. Contact your local dealer for service
- Disinfect the upholstery. please check the disinfection section in this User Guide for more information
- Inspect the detachable parts. If required, disinfect the parts with a 70% disinfectant solution. It is recommended to wipe of any residue and dirt using a cloth wrung in warm water and a mild detergent/soap without chlorine. Let the detachable parts dry before disinfection

After every use

- Wipe the product dry

Daily

- Visual inspection. Repair or replace any damaged or worn-out parts
- Ensure that all fasteners are in place and fitted correctly
- Check all fittings and buckles for signs of worn out parts
- Be aware of any MAX mark indications before adjustment

Monthly

- Use a cloth with warm water and a mild detergent to clean the product
- All nuts and bolts on this product must be checked and tightened to avoid unnecessary failures
- Oil the swing-away parts. We recommend you use a professional lubricant system

Annually

- Inspect the frame for cracks or signs for worn-out parts and carry out annual service. Never use a product which appears to be substandard or faulty
- Inspect the piston rods are greased, extract the piston rods to check grease

Warewashing machine

This product can be washed at:

- 60°C in 10 minutes
- 85°C in 3 minutes

.. using a mild detergent in a washing machine designed for medical devices. Use the machine's drying function to dry the product. Clean the accessories separately.

Before washing

Avoid degreasing of the piston rods, the actuators should be retracted to minimum stroke and without load before washing.

- Remove all accessories, unplug all cables and make sure to dismount the battery, control-box and handset. Leave the actuators with the cables mounted on the product (see page 22)

After washing

- Ensure correct placement of cables inserted into the control box when reinstalling the power unit system (see page 23)
- Check that the handset functionalities works correct

Disinfection

The product can be disinfected with a 70% disinfectant IPA solution. It is recommended to wipe off any residue and dirt from the product, using a cloth with warm water and a mild detergent/soap without chlorine and let it dry before disinfection.

Service interval

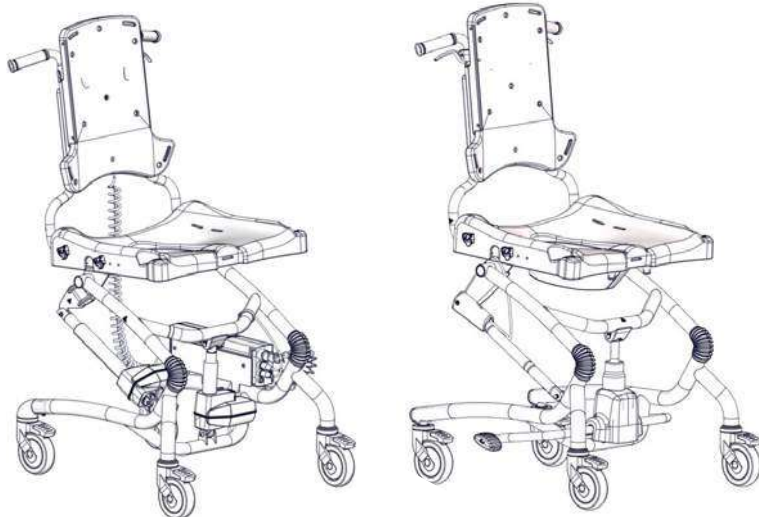
A detailed inspection of the product must be made every 12 months (every 6 months for heavy users) and every time the product is reissued for use. The inspection should be carried out by a technically competent person who has been trained in the use of the product.

The lifetime of this product in normal use is 5 years if all maintenance and servicing is carried out in accordance with the manufacturer's instructions and demonstrably recorded.

Service Check List:

Check and adjust the following functions as per their intended use. Repair or replace any damaged or worn-out parts. Please check the:

- Cushioning
- Products equipped with a hydraulic pump for height adjustment. Frame height can be adjusted up and down
- Powered versions are able to retain power after charging the battery
- For powered versions, reset the power unit system using the handset by pressing the up(f) and down(g) buttons simultaneously for 5 seconds
- Products equipped with power for height adjustment. The cables are in good condition and the handset functionalities for height adjustment works as intended.
- Nuts and bolts on the product are tightened
- Oil the swing-away parts
- Fittings and buckles for signs of worn out parts
- Product for cracks or signs for worn-out parts
- All labels on the product are intact

Power Unit system

Handset functionality

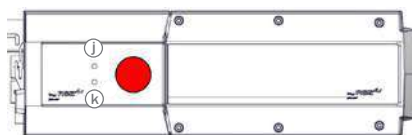
- Battery is OK, no need for charging. Green LED lights when activating functions
- Battery is soon empty, the battery needs recharging. Orange LED Lights when activating functions
- The battery needs recharging, red LED flashes when activating functions. It is still possible to lower the seat height and to tilt the seat forwards
- Service required, LED Lights. Check that all connectors are firmly fixed in the connections. If the LED continues to light up see Service interval page 34 and go through the Service check list on page 35.
- Overload, orange LED flashes when trying to heighten the seat level. Lower the seat until stop to reset the system
- Increase the seat height
- Lower the seat height
- Tilt the seat forward
- Tilt the seat backward

Troubleshooting:


Make sure that the emergency stop is not activated. In case of the handset malfunctioning the seat height can be adjusted by using the control box. Using a pen or a 3mm Allen Key, activate the up and down points in the control box.

- Increase the seat height
- Lower the seat height

Recharging the battery:

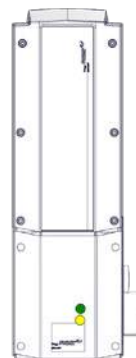- Connect the charger to mains. The charger indicates whether the charger is connected to the mains (green LED)
- Place the battery in the charger, battery is being charged (yellow LED)
- Charging time: approximately 4-5 hours

Warning

Caregiver

- Do not operate the Power unit system unless you have received training in using it
- Do not leave children and youngsters alone with the handset
- Do not let children and youngsters play with the handset
- Do not leave the handset within reach of children and youngsters.
- It is important to remove the transport plastic bag before using the handset. Cables need to be placed in such a way that users or carers cannot trip and injure themselves.

Product

- The control box has an emergency stop
- The system must not be sprayed directly with a highpressure cleaner
- Cleaning with a steam cleaner is not permitted
- Do only use the power unit within specified working limits
- It is recommended to check the handset and cable for damage and holes caused by violent handling at least once a year
- Do not submerge the handset in water

Technical Data

Battery: BAJ1 - IPX5

- Capacity 2.9 Ah - 24V DC
- Duty cycle: Max. 10% or 2 min. use then 18 min. without use

The batteries have a maximum life of 4 years but this can vary depending upon the conditions of use and charging practices

Control box: CBJC - IPX4

- Output voltage 24V DC

Battery charger: CHJ2 - IPX5

- Mains: 100 - 240VAC/50-60 Hz switch mode
- Charging current: Max. 650 mA

Actuator: LA40 - IPX6

- Motor: 24-33 V DC, Standard motor
- Noise level: ≤ 50 db (A)

Handset: HB 30 IPX4

Detachable parts

Handset - HB 30

- Battery - BAJ1
- Control box - CBJC

Applied part

- Handset - HB 30

IP-Rating table

First Digit: Solids

The first digit indicates the level of protection that the enclosure provides against access to hazardous parts (e.g., electrical conductors, moving parts) and the ingress of solid foreign objects

IP0X	No protection against contact and ingress of objects
IP1X	Any large surface of the body, such as the back of the hand, but no protection against deliberate contact with a body part.
IP2X	Fingers or similar objects.
IP3X	Tools, thick wires, etc.
IP4X	Most wires, screws, etc.
IP5X	Ingress of dust is not entirely prevented, but it must not enter in sufficient quantity to interfere with the satisfactory operation of the equipment; complete protection against contact.
IP6X	No ingress of dust; complete protection against contact.

Ambient and surface temperature

The table below illustrates how long time it takes for the surfaces of the Heron to reach an ambient temperature of 20 °C from -10 °C and +50 °C.

Surface temperature			
-10 °C	to	+20 °C	108 min
+50 °C	to	+20 °C	129 min

Environmental conditions

- Operating temperature: +5 °C to +40 °C
- Relative humidity: 20% to 90%
- Atmospheric pressure: 800 to 1060 hPa
- Storage temperature: -10 °C to +50 °C

Second Digit: Liquids

Protection of the equipment inside the enclosure against harmful ingress of water.

IPX0	No protection
IPX1	Protection against vertically falling drops of water e.g. condensation
IPX2	Protection against direct sprays of water up to 15° from the vertical
IPX3	Protected against direct sprays of water up to 60° from the vertical
IPX4	Protection against water sprayed from all directions - limited ingress permitted
IPX5	Protected against low pressure jets of water from all directions - limited ingress
IPX6	Protected against temporary flooding of water, e.g. for use on ship decks - limited ingress permitted
IPX7	Protected against the effect of immersion between 15 cm and 1 m
IPX8	Protects against long periods of immersion under pressure

Electromagnetic compatibility (EMC) environment

This equipment is for use in home healthcare facility environment.

This equipment is not likely to cause any harmful interference in nearby equipment; however, it can be impaired by electromagnetic (EM) interference emitted by other equipment, which can lead to malfunctions. Do not use this equipment where there is a risk that other electrical and/or electronic equipment can interfere with this equipment.

This equipment complies with the following standards:

IEC60601-1-2:2014 Medical electrical equipment - Part 1-2: General requirements for basic safety and essential performance - Collateral Standard: Electromagnetic disturbances - Requirements and tests (EN 60601-1-2:2015)

EN 12182:2012, Clause 7, Assistive products for persons with disability – General requirements and test methods

EMC - WARNINGS

Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.

Use of accessories, transducers and cables other than those specified or provided by R82 could result in increased EM emission or decreased EM immunity of this equipment and result in improper operation. Several power plugs are supplied with this equipment. Please use the correct power plug suitable for your country. Please chose among the supplied power plug adaptor units: English, Australian, EU, US/Japanese plugs.

Portable radio frequency (RF) communication equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the heron w/power, including cables specified by R82. Otherwise, degradation of the performance of this equipment could result.

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Test modes during emission tests:

1. Movement mode (battery). Actuator running in/out
2. Charging mode, actuator not running.

EM emission standards	Classification/levels
Conducted and radiated RF emission CISPR 11	Group 1 (the product uses RF energy only for its internal function) Class B (the product is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes)
Harmonic distortion IEC 61000-3-2	Class A. No testing performed, as the rated power for this equipment is less than 75 W. No limits are specified for equipment with a rated power of 75 W or less.
Voltage fluctuations and flicker IEC 61000-3-3	Complies

Test modes during immunity tests:

1. Battery mode, standby armed/ready.
2. Charging mode, actuator not running.

EM immunity standards	IEC 60601-1-2:2014 test levels	Additional test levels
Electrostatic discharge IEC61000-4-2	± 8 kV contact ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV air	± 6 kV contact
Radiated RF EM fields IEC 61000-4-3	3V/m (Professional) 10 V/m (Homecare) 80 MHz – 2.7 GHz 80 % AM at 1 kHz	20 V/m 800 MHz – 2.5 GHz 80 % AM at 1 kHz
Proximity fields form RF wireless communication equipment IEC 61000-4-3	IEC 60601-1-2:2014 Table 9	N/A
Electrical fast transients / bursts IEC 61000-4-4	± 1 kV, ± 2 kV 5 kHz repetition frequency	N/A
Surges, Line-to-line IEC 61000-4-5	$\pm 0,5$ kV, ± 1 kV, ± 2 kV	N/A
Conducted disturbance induced by RF fields IEC 61000-4-6	6 Vrms 0.15 MHz - 80 MHz 80% AM at 1 kHz 0.15 MHz - 80 MHz 80% AM at 2 kHz	N/A
Rated power frequency magnetic fields IEC 61000-4-8	30 A/m 50 Hz and 60Hz	N/A
Voltage dips, short interruptions and voltage variations IEC 61000-4-11	0 % UT for 0,5 cycle at 0°, 45°, 90°, 135°, 180°, 225°, 270° and 315° phase angles 0 % UT for 0,5 cycle 0 % UT for 1 cycle 70 % UT for 25 cycles (50 Hz) 70 % UT for 30 cycles (60 Hz) 0 % UT for 250 cycles	40 % UT for 5 cycles

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© 2017 R82 A/S All rights reserved The R82 logo and the icons are registered trademarks of R82 A/S
R82 A/S